



## How To Insert Your CADpro<sup>TM</sup> Temps Restoration

Your CADpro<sup>TM</sup> Temps restoration is CAD/CAM milled from industrially produced PMMA. CADpro<sup>TM</sup> Temps are very strong (~140MPa) and are indicated for intraoral use for up to one year, making CADpro<sup>TM</sup> Temps ideal for all situations requiring long-term provisionalization.

### Steps to prepare your CADpro<sup>TM</sup> Temps for Insertion:

1. Prior to inserting the case, try-in the CADpro<sup>TM</sup> Temps restoration intraorally to ensure it seats fully.
2. Sandblast the internal aspect of the CADpro<sup>TM</sup> Temps abutment copings/retentive wings to increase retention of the reline material/cement.
3. Reline the appropriate areas of the CADpro<sup>TM</sup> Temps with either bis-acryl, flowable composite, or acrylic.
4. Trim excess from margins, adjust occlusion, and polish.
5. Insert the relined CADpro<sup>TM</sup> Temps restoration with a conventional temporary cement such as Telio CS<sup>\*</sup>, Temp Bond NE<sup>\*</sup>, Durelon<sup>TM</sup>.



BY Jurim  
DENTAL STUDIO

T: 1-516-466-1722  
F: 1-516-466-1261

Dr. Name \_\_\_\_\_

Acct. # \_\_\_\_\_ Phone # \_\_\_\_\_

E-mail \_\_\_\_\_

Address \_\_\_\_\_

Patient Name \_\_\_\_\_  Female  Male

Deliver by \_\_\_\_\_  Call before starting case

**Rx**

All Restorations  
Made in the USA

Signature \_\_\_\_\_ License # \_\_\_\_\_

© 2014 Jurim Dental Studio. See our website [www.CADproDental.com](http://www.CADproDental.com) for warranty terms and conditions.

*For more information visit us @*

[www.JurimDental.com](http://www.JurimDental.com) | [www.CADproDental.com](http://www.CADproDental.com) | 516.422.1722



- Biocompatible, High Strength PMMA
- Ideal for Long-Term Provisionalization
- Inexpensive and Faster Production Turn-Around Time
- Custom Stained and Finished for Unparalleled Aesthetics

